



English

Gateway Placement Instructions

For use with the LIFEPAK® 12 Defibrillator/Monitor and the LIFEPAK 15 Monitor/Defibrillator

1. Place Gateway in outer pocket of side pouch as shown.
2. Position Gateway so the cable exits toward the rear of the pouch.
3. Connect cable to port on back of the monitor. If using bag for the LIFEPAK 15 device, secure cable using Velcro® flap.
4. **DO NOT STORE GATEWAY IN REAR POUCH.** The Gateway may interfere with battery operation and cause unexpected shutdown of the monitor.