PHILIPS

Wireless Link

ENGLISH

Introduction

This document is an addendum to the *HeartStart MRx Instructions for Use*. It explains how to use the features of the HeartStart MRx's Wireless Link Option.

The Wireless Link Option allows the HeartStart MRx monitor/defibrillator to wirelessly transmit data to Philips' Telemedicine System, Data Messenger, ER Pro or to third-party electronic patient care record (ePCR) software. Using Wireless Link, you can send:

- data (12-Leads, periodic clinical data) to the Telemedicine System via a cellular broadband connection or a gateway via WiFi.
- Event Summaries to ER Pro or a third-party ePCR using Data Messenger via WiFi.

WARNING: Before using the HeartStart MRx with Wireless Link Option, you should familiarize yourself with the operation of the HeartStart MRx. Refer to the *HeartStart MRx Instructions for Use*.

CAUTION: Removing all power (AC and battery) from the HeartStart MRx when the device is still on or shortly after it has been turned off may disable Wireless Link and require an extensive repair. After turning the HeartStart MRx off, wait approximately 15 seconds for the Wireless Link to complete its shutdown before removing all power.

Setup, Configuration and Connection

Before you can send data, Wireless Link, the HeartStart MRx and the system receiving your data must be set up and configured properly by your IT professionals. For set-up and configuration instructions please visit: <http://incenter.medical.philips.com/default.aspx?tabid=730>. Click on the HeartStart MRx link and then the Implementation Guides dropdown to find Philips' *HeartStart MRx Data Transmission Implementation Guide* (English only) - instructions on how to configure the HeartStart MRx, Wireless Link and your data management software properly.

NOTES: All configurations need to be completed before using the Wireless Link option. Failure to do so results in an inability to use the Wireless Link option.

When the HeartStart MRx has a low battery, only vital functionality directly related to patient care is available. Wireless Link does not work when the device has a low battery.

Connecting Wireless Link

Once your Wireless Link, HeartStart MRx and receiving system have been set-up and configured properly to send data to the appropriate destinations, you need to connect the Wireless Link to the HeartStart MRx.

☉ To connect the Wireless Link to the HeartStart MRx:

- 1 Depending on your carry case, place the Wireless Link device in the carry case as shown in Figure 1. There are two recommended locations:

- in the back pouch
- in the therapy side pouch


Figure 1 Recommended Wireless Link Placement

Black Soft Carry Bag - back pouch

Place Wireless Link as shown in the meshed bag along the back of the back pouch. Thread the connectors through the larger of the two holes to the HeartStart MRx.

Black Soft Carry Bag -
Therapy side pouch with elastic band

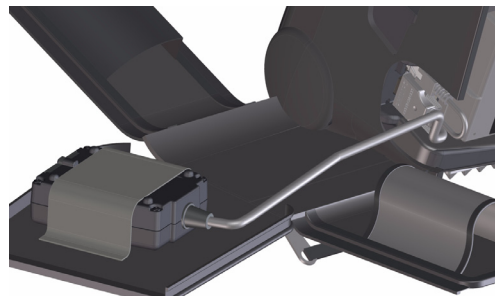Place Wireless Link face down as shown in elastic band on the therapy side pouch wall. Thread cable along bag bottom, through the bottom opening and connect to the HeartStart MRx.

Red Carry Case - therapy side pouch

Place Wireless Link as shown in the therapy side pouch. Rest it on the bottom of the bag. Thread cable along bag wall, behind dividers, through the opening to the HeartStart MRx.

Black Soft Carry Bag -
Therapy side pouch without elastic band


Rest the Wireless Link on the bottom of the far outside pouch, facing out. Thread cable along bag bottom, then up through the Therapy port opening and connect to the HeartStart MRx.

CAUTION: Placing the Wireless Link in positions other than the ones shown in Figure 1 may result in diminished signal strength.

- 2 With the HeartStart MRx off, plug the RS-232 serial connector into the RS-232 serial port on the back of the HeartStart MRx. Confirm the plastic stabilizer is in place and screw in the pins. See Figure 2.
- 3 Plug the LAN cable into the LAN port on the back of the HeartStart MRx. See Figure 2.
- 4 Turn the HeartStart MRx on. The Wireless Link automatically turns on and the option is enabled.

Figure 2 **Connecting Wireless Link**

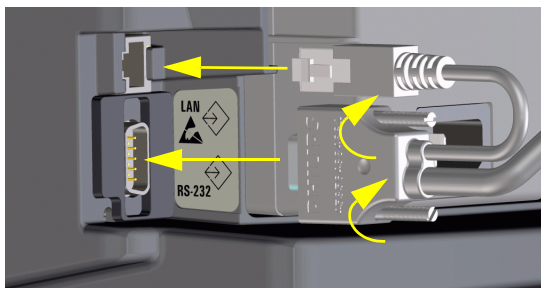CAUTION: To test your ability to successfully transmit before using in a clinical situation, you must create sample data and send it to a live Telemedicine System or Data Messenger. Be sure to delete the data from your Telemedicine System or Data Messenger after the test is complete so as to not confuse it with real patient data.

Sending Data with Wireless Link

You can use Wireless Link to send data during patient care or post-event. During an event, you can send 12-Leads or periodic clinical data (PCDT). Post-event transmissions include Event Summaries. For more information on sending data, see the Data Transmission chapter in the *HeartStart MRx Instructions for Use*.

NOTES: Assuming Wireless Link is connected when the HeartStart MRx is first powered on, it typically takes approximately 90 seconds for Wireless Link to be ready to send data. Different IT environments and configurations can take longer. If you initiate a 12-Lead transmission in a clinical mode before Wireless Link is ready, the HeartStart MRx automatically retries to send data in 30 seconds.

Successful transmission is dependent on the availability of public or private telecommunication networks and other influencing factors, particularly for cellular communications. These factors include but are not limited to weather, geography, your transmitting location, cellular service availability, the number of cellular service users in the area, and your authorization/subscription from the cellular service provider. Data transmission via the telecommunication networks cannot be assured with the use of such networks. Make sure you have a contingency plan for interrupted or unsuccessful data transmissions.

The steps to send data from the HeartStart MRx are the same regardless if data is being sent via WiFi or cellular broadband communication.

© To send the current 12-Lead from 12-Lead Mode during an event:

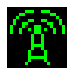
- 1 In 12-Lead Mode, press the Menu Select  button.
- 2 Using the Navigation buttons, select **Send 12-Lead** and press the Menu Select button.
- 3 Using the Navigation buttons, select the pre-configured destination for the 12-Lead and press the Menu Select button. Transmission begins.


◎ To send periodic clinical data from clinical mode during an event:

- 1 Confirm the HeartStart MRx is in a clinical mode and press the Menu Select button.
- 2 Using the Navigation buttons, select **Start Data Transmit** and press the Menu Select button.
- 3 Using the Navigation buttons, select the pre-configured Telemedicine destination for the data and press the Menu Select button. Transmission begins.

When the Wireless Link Option sends periodic clinical data, the HeartStart MRx displays an icon indicating connectivity status. See Figure 3.

Figure 3 Wireless Link PCDT Connectivity Icons

Symbol	Definition
No symbol	A transmission has not been initiated.
After a transmission has been initiated:	
	A Periodic Clinical Data Transmission has been initiated. A network connection may not be available. (White symbol)
	Wireless Link is currently transmitting PCDT data. (Green symbol)
	Wireless Link started a PCDT transmission but the connection failed or a transmission was initiated before the Wireless Link device was ready.

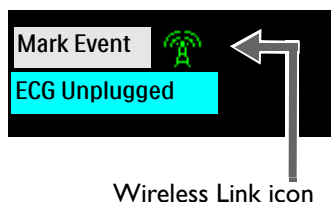◎ To send Event Summaries from Data Management Mode, post-event:

- 1 Confirm the HeartStart MRx is in Data Management Mode, Data Messenger is running on a receiving computer and Wireless Link is connected. The HeartStart MRx automatically communicates with Data Messenger (**Connecting** appears on the upper left part of the display) and when connected, brings up a menu to send a single event or all events. (Depending on your network configuration, this connection may take some time to be established. An average connection time is in the 30-75 second range.)

If you want to transfer a single event:


- 2 Use the **[Prev Item]** and **[Next Item]** softkeys to select the desired Event Summary to download.
- 3 Using the Navigation buttons select **Send Selected** and press the Menu Select button. See Figure 4.
- 4 Transmission begins. Once the transmission is complete a note appears on the screen. You need to acknowledge it to continue.

If you want to transfer all events:

- 2 Using the Navigation buttons select **Send All** and press the Menu Select button. See Figure 4.
- 3 Transmission begins. Once the transmission is complete a note appears on the screen. You need to acknowledge it to continue.

NOTE: Sending all Event Summaries also erases each one from Data Management Mode. If you want to keep other copies, copy the files to an external data card before transfer or send them each individually

Figure 4 **Sending files in Data Management Mode**

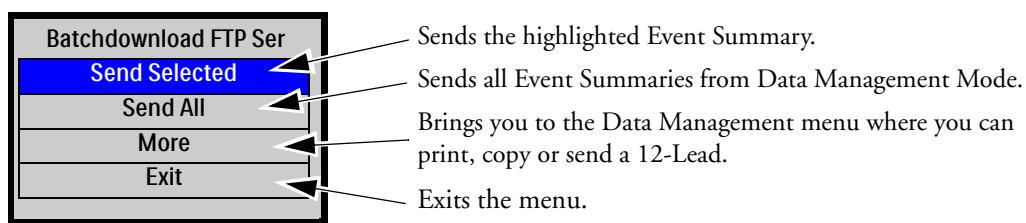NOTES: If the Wireless Link is in the middle of transmitting data and additional transmissions are initiated, the newly requested transmissions are sent in order of request after the first transmission is finished.

If you exit a clinical mode during a transmission or a transmission error occurs, all current and pending PCDT transmissions are cancelled. 12-Leads complete their transmission, if possible.

For more information on tracking data transmissions and cancelling a transmission, see the Data Transmission Chapter in the *HeartStart MRx Instructions for Use*.

When multiple wireless networks are available, confirm with your IT professional how to connect to the correct WiFi network, thereby assuring your Event Summaries arrive at the proper destination.

All transmitted personal patient-identifiable data is secured to ensure patient confidentiality. Handle in accordance with HIPAA or your local patient privacy requirements.

Troubleshooting

See the following sections to troubleshoot issues related to the Wireless Link device. For more data transmission troubleshooting information, see the *HeartStart MRx Instructions for Use*.

Wireless Link Indicators

While troubleshooting the Wireless Link, it is often beneficial to look at the various indicator lights located on the side of the Wireless Link Device. See Figure 5 for definitions of each indicator.

Figure 5 **Wireless Link Indicators**

Indicator	On - Solid Light	Off - No Light	Flashing Fast	Flashing Slow
Cellular Network	Connected - Ready for data transfer	Cellular option is powered off	Searching for subscribed cellular network	Subscribed cellular network found. Waiting for IP configuration
WiFi Network	Connected - strong signal	WiFi option powered off or no connection	Connected - fair signal	Connected - weak signal
Ethernet	Connected to the HeartStart MRx	Not connected to the HeartStart MRx or it is powered off	Data transfer in progress	Wireless Link signalling HeartStart MRx
Run	Wireless Link in Bridge or Router Mode	Wireless Link in configuration mode	Wireless Link is in Access Point Mode	
Power	Wireless Link powered on	Wireless Link off		

NOTE: Depending on your particular configuration, some of these lights may or may not be applicable to your transmission. Talk with your IT professional to find out which lights you should be concerned about and if you experience problems transmitting data.

Symptoms and Messages

Table 1 provides troubleshooting tips for the Wireless Link. For more data transmission troubleshooting information, see the *HeartStart MRx Instructions for Use*. If your Wireless Link does not work properly and none of the following troubleshooting tips solve the problem, contact your IT professional.

Table 1 Troubleshooting

Symptom	Possible Cause	Possible Solutions
Wireless Link Unplugged INOP message	Your Wireless Link device is not connected to the HeartStart MRx.	Wait up to 45 seconds from when the HeartStart MRx was powered on for the Wireless Link and HeartStart MRx to complete their data connection.
	There has been a failure in the Wireless Link device or its cable.	Check that RS-232 and LAN connections are secure. Check the power light on the side of Wireless Link. If the light is not on, the device or cable may be broken. Replace Wireless Link or cable assembly. If the light is still not on, call for service. If the power light is on and checking connections does not solve the problem, replace the Wireless Link or call for service.
Wireless Link Disabled: DoS INOP message	The HeartStart MRx has detected an unauthorized denial of service data attack (DoS) and shut the Wireless Link down.	Turn the HeartStart MRx off for at least 10 seconds then restart. If the problem persists, contact your IT professional.
Transmission Failed. Automatic Retry in xx Seconds message	During a 12-Lead transmission in Clinical Mode, Wireless Link has lost its data connection.	Allow the device to continue trying to send its transmission. The countdown is 30 seconds long. If you want to retry immediately, cancel your transmission from 12-Lead Mode and attempt to resend. Change locations to improve signal strength.
Your transmission is taking a long time	There is a weak transmission signal or the Wireless Link is incorrectly positioned.	Check orientation of Wireless Link and correct positioning if required. Change your location to improve signal strength.
You are trying to send a 12-Lead but there is no icon on the screen	This is normal operation. Wireless Link connection icons do not appear on the HeartStart MRx while sending 12-Leads. The 12-Lead transmission progress bar is the only indication of the transmission's progress.	No action required.

NOTE: See the Data Transmission chapter in the *HeartStart MRx Instructions for Use* for more information on data transmission messages and troubleshooting.

Other Considerations

Configuration Options

Most of the time, your IT professional selects the Configuration choices for Wireless Link. Tables 2 and 3 list the additional Wireless Link configuration items available in Configuration Mode. See the Configuration chapter in your *HeartStart MRx Instructions for Use* for additional Configuration Mode information. The default settings are listed in bold.

Table 2 **Data Transmission - Transmit Devices Settings Options**

Parameter	Description	Setting Choices
Bluetooth	Set this option to On if you have a Bluetooth card installed and want to use it to transmit data from the HeartStart MRx.	On, Off
Wireless Link	When the Wireless Link option is available, it determines whether the HeartStart MRx looks for the Wireless Link device.	On , Off
If both on in Clinical Mode	Selects the default Clinical Mode transmission method when the HeartStart MRx has both Wireless Link and Bluetooth transmission options installed and set to On .	Wireless Link , Bluetooth

Table 3 **Data Transmission - Wireless Link Settings Options**

Parameter	Description	Setting Choices
Access Point	Determines whether the Wireless Link operates in Access Point Mode when entering Data Management Mode. (This is separate from a similar configuration option on the Wireless Link device itself.)	Yes, No
http Proxy Address	If your facility has a Proxy Address, it may be configured here.	15 characters, blank
http Proxy Port	If your facility has a Proxy Port, it may be configured here.	15 characters, blank
Wireless Link Address	Sets the Wireless Link's IP address. Change this only when your network already has the default address in use. (A corresponding change must be made in the Wireless Link module as well).	192.168.171.2 , nnn.nnn.nnn.nnn

Cleaning

Wireless Link can be cleaned using the same cleaning instructions for your HeartStart MRx monitor/defibrillator listed in the Maintenance chapter of the *HeartStart MRx Instructions For Use*.

Event Summary

If the Wireless Link option is installed, Wireless Link INOPs appear in the Event Summary.

Operational Check

If the Wireless Link option is installed, your HeartStart MRx checks the connection and Wireless Link functionality during an Operational Check. Connect the Wireless Link to the HeartStart MRx before running Operational Check.

Results are displayed on the screen. **Pass** is displayed if the test is successful. If the Operational Check fails the Wireless Link part of the test or if Wireless Link is not connected at all, **Fail/Check Cables and Module** is displayed. Check that the cables are securely plugged in and re-run Operational Check.

Inspect the Wireless Link device (cables and body) for visible signs of damage as part of the manual User Checks after completing an Operational Check.

Battery Capacity

The HeartStart MRx uses a M3538A Lithium Ion Battery. A single, new, fully-charged battery, operating at room temperature 25°C (77°F) with a cellular signal available that has an RSSI better than -80dB, provides enough energy for at least 4.5 hours of operation with ECG, SpO₂, CO₂, monitored continuously, NBP measured every 15 minutes, 20 200J discharges, 4 12-Leads acquired and transmitted, and PCDT data transmitted every 4 minutes via Wireless Link.

Specifications

Weight: 0.8 lbs.

Dimensions: 120 mm × 90 mm × 45 mm (4.72 in. x 3.54 in. x 1.77 in.)

Cable Length: 352mm (13.85 in.)

Temperature: 0°C to 45°C operating; -20°C to 70°C storage

Humidity: Up to 95% relative humidity

Cellular transmission (2G and 3G) -

Wireless Link supports PAP, CHAP and PAP+CHAP for Username/Password authentication

Supported Frequency Bands:

GSM/GPRS/EDGE: 850/900/1800/1900 MHz

UMTS/HSDPS/HSUPA: 800-850/900/1900/2100 MHz and AWS band (1700/2100 MHz) (B1, B2, B4, B5, B8)

CDMA 1xRTT/EV-DO rev0/EV-DO revA: 800/1900 (BC0, BC1)

Maximum RF output power:

Power Class 4 (2 W, 33 dBm) for GSM/GPRS 850/900 MHz bands

Power Class 1 (1 W, 30 dBm) for GSM/GPRS 1800/1900 MHz bands

Power Class E2 (0.5 W, 27dBm) for EDGE 850/900 MHz bands

Power Class E2 (0.4 W, 26 dBm) for EDGE 1800/1900 MHz bands

Power Class 3 (0.25 W, 24 dBm) for UMTS 850/900/1900/2100 MHz bands

Power Class 3 (0.25 W, 24 dBm) for 1xRTT and EV-DO

WiFi Transmission -

ISM Band: 802.11b and 802.11g use the 2.4 GHz ISM band

Transmit Power: ± 15 dBm typical for both 802.11b and 802.11g

Baseline Modulation: OFDM (802.11g); DSSS/CCK (802.11b)

Security and Encryption: WEP (64/128 bit), WPA-PSK (TKIP/AES), WPA2-PSK (AES/TKIP), WPA-ENT (PEAP-MSCHAPV2) and WPA2-ENT (PEAP-MSCHAPV2)

NOTE: The HeartStart MRx, as a system, is designed for international regulatory compliance. It is properly registered, where required, in all countries where sales and distribution is designated.

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Printed in the U.S.A. February 2016