

**Ositech Titan II Gateway
Placement Guide
LIFEPAK® 12 Defibrillator/Monitor &
LIFEPAK® 15 Monitor/Defibrillator**

Prior to housing your Titan II Gateway, please ensure that your Titan II Gateway is configured for your networking environment.

The Titan II Gateway is housed within a plastic protective shell, which houses the Titan II Gateway, the antenna, and connection cable for your LIFEPAK monitor. The protective shell is designed to fit the LEFT SIDE PROTECTIVE POUCH of either the LIFEPAK 12 Defibrillator/Monitor or the LIFEPAK 15 Monitor/Defibrillator. This placement will result in the ideal antenna position for optimal results.



Step 1 (Applicable to all LIFEPAK Device Models)

Place the Titan II Gateway in the LEFT SIDE POUCH, such that the indicator lights are facing upwards and the connection cable exits towards the rear of the protective pouch.

Step 2 (For LIFEPAK 15 Monitor/Defibrillators ONLY)

Route the connection cable underneath the Velcro flap out the rear of the protective pouch. The LIFEPAK 12 Defibrillator/Monitor pouch does not have this flap. Secure the Velcro flap(s).



Step 3 (Applicable to all LIFEPAK Device Models)

Insert the connection cable from the Titan II Gateway to the Hypertronic Connector on the rear of the LIFEPAK monitor. The cable incorporates a 90 degree connector which makes the connection to the LIFEPAK monitor's left side pouch very convenient, and secure.



The Titan II Gateway should only be placed in the side pouch of your LIFEPAK monitor. If your device is equipped with a back pouch, DO NOT PLACE MODEM IN BACK POUCH, as this may cause interference.